

PERFORMANCE MEETS SCIENCE™



Tape

COMPLETE KNEE



WATCH THE VIDEO

www.gotape.com/completeknee



www.facebook.com/MYGOTAPE



www.twitter.com/GOTAPE



1 Place knee at 90 degrees.



Clean skin with soap and water



Apply at least 30 minutes prior to activity



Clip hair to 1 cm or less (if needed)



Stretch the skin to be taped



Do not stretch the ends of the tape



Rub entire application well after applying each strip




2 Anchor middle portion of half-strip under knee with 90% stretch.



3 Lay each end down with no stretch.



4 Repeat with second half-strip just above the knee.

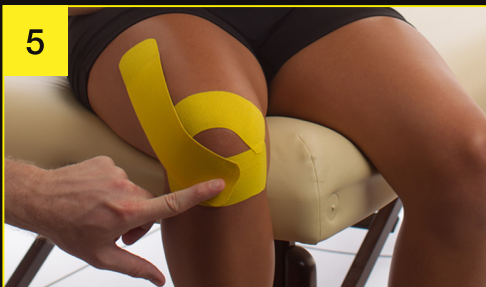

5 Anchor full-strip on side of thigh with no stretch.



6 Apply to top of knee with 25% stretch.



7 Apply across lower part of knee with 50% stretch.



8 Lay end down with no stretch.



9 Repeat steps 5-8 for other side of thigh.



10 Rub application well to securely fix all edges to skin.